

# MD DrugScreen

## TEST CUP



### PRODUCT FEATURES:

- CLIA-Waived drug test cup
- Fast, dark test lines
- Negative results within 60-90 seconds
- Non-negative results in 5 minutes
- Donor friendly cup opening
- Results can be photocopied
- Optional test for adulterants



\* For Forensic Use Only

CONTACT US at  
[Sales@MedicalDisposables.us](mailto:Sales@MedicalDisposables.us)

### MD DrugScreen Test Cup configuration:

<b>Amphetamine</b> 500 & 1,000 ng/mL	(AMP)	<b>Methamphetamine</b> 500 & 1,000 ng/mL	(MET)
<b>Barbiturates</b>	(BAR)	<b>Morphine</b>	(MOR)
<b>Buprenorphine</b>	(BUP)	<b>Opiates</b> 300 ng/mL	(MOP)
<b>Benzodiazepine</b>	(BZO)	<b>Opiates</b> 2,000 ng/mL	(OPI)
<b>Cocaine</b> 150 & 300 ng/mL	(COC)	<b>Oxycodone</b>	(OXY)
<b>Ecstasy</b>	(MDMA)	<b>Phencyclidine</b>	(PCP)
<b>Marijuana</b>	(THC)	<b>Propoxyphene</b>	(PP)
<b>Methadone</b>	(MTD)	<b>Tricyclic Antidepressants</b>	(TCA)
<b>Methadone Metabolite</b>	(EDDP)		



(888) 863-1112

6831 Edgewater Commerce Pkwy Ste 1101  
Orlando, FL 32810

[MedicalDisposables.us](http://MedicalDisposables.us)  
[Sales@MedicalDisposables.us](mailto:Sales@MedicalDisposables.us)



HAVE QUESTIONS? PLEASE CALL US (888) 863-1112

# MD DrugScreen Test Cup

## Procedure Card

### STEP 1

Have the donor provide a specimen and return the cup to the collector.



### STEP 2

Replace and tighten the cap. Verify the temperature is within the range on the temperature strip. (90° - 100°F) (32° - 38°C)



°F90	2	4	6	8	100
GREEN=TEMP / TAN=-1°F / .5°C					
°C32	3	4	6	7	38

### STEP 3

Remove the label to view the results.



### STEP 4

If equipped, place the adulteration card next to the adulterant strip on the cup. If any of the test pads match any of the colors on the left, adulteration is indicated.

	Normal	Abnormal
OXI	Negative	
S.G.	1.003 1.005 1.015 1.025	1.000  >1.35
pH	4.0 7.0 9.0	2.0 3.0  10.0 11.0 12.0
NIT	20mg/dl  Negative	50mg/dl  100mg/dl
GLUT	Negative	
CREA	100mg/dl  20mg/dl	0mg/dl  0mg/dl

### STEP 5


Read the results when all upper "Control" or "C" lines have appeared. Negative results can be read as soon as all "Test" or "T" lines are visible.



(Wait 5 minutes to determine a Positive Result)

#### INTERPRETING RESULTS

##### NEGATIVE

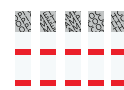


A negative result is indicated by the presence of the lower "Test" or "T" line of each designated drug.

##### NEGATIVE



The presence of even a very light "Test" or "T" line indicates a negative result.

##### POSITIVE

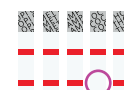

A positive result is indicated by the absence of the lower "Test" or "T" line for a specific drug. Wait 5 minutes to determine a positive result.

##### INVALID



An invalid result is indicated by the absence of the upper "Control" or "C" line and the lower "Test" or "T" line for a specific drug. If this occurs run a second test.