

# MD DrugScreen TEST CUP



MedicalDisposables.us

## PRODUCT FEATURES:

- Soft plastic materials with the same quality and accuracy
- CLIA-Waived drug test cup
- Fast, dark test lines
- Negative results within 60-90 seconds
- Non-negative results in 5 minutes
- Donor friendly cup opening
- Results can be photocopied
- Optional test for adulterants



\* For Forensic Use Only

Item Code: MDC-6164EFKT

## MD DrugScreen Test Cup configuration:

<b>Amphetamine</b> 500 & 1,000 ng/mL	(AMP)	<b>Methadone</b>	(MTD)
<b>Barbiturates</b>	(BAR)	<b>Methadone Metabolite</b>	(EDDP)
<b>Buprenorphine</b>	(BUP)	<b>Methamphetamine</b> 500 & 1,000 ng/mL	(MET)
<b>Benzodiazepine</b>	(BZO)	<b>Opiates</b> 2,000 ng/mL	(OPI)
<b>Cocaine</b> 150 & 300 ng/mL	(COC)	<b>Oxycodone</b>	(OXY)
<b>Ecstasy</b>	(MDMA)	<b>Phencyclidine</b>	(PCP)
<b>Alcohol</b>	(ETG)	<b>Synthetic Marijuana</b>	(K2)
<b>Fentanyl</b>	(DFN)	<b>Tramadol</b>	(TRA)
<b>Marijuana</b>	(THC)	<b>Tricyclic Antidepressants</b>	(TCA)



Like Us!



Follow Us!



Follow Us!



☎ (888) 863-1112

📍 6831 Edgewater Commerce Pkwy Ste 1101  
Orlando, FL 32810

🌐 MedicalDisposables.us  
✉ Sales@MedicalDisposables.us

**HAVE QUESTIONS? PLEASE CALL US (888) 863-1112**

# MD DrugScreen Custom Cup Procedure Card

## STEP 1

Have the donor provide a specimen and return the cup to the collector.



## STEP 2

Replace and tighten the cap. Verify the temperature is within the range on the temperature strip. (90° - 100°F) (32° - 38°C)



## STEP 3

Remove the label to view the results.



## STEP 4

If equipped, place the adulteration card next to the adulterant strip on the cup. If any of the test pads match any of the colors on the left, adulteration is indicated.

	Normal	Abnormal
OXI	Negative	
S.G.	1.003 1.005 1.015 1.025	1.000  >1.35
pH	4.0 7.0 9.0	2.0 3.0  10.0 11.0 12.0
NIT	20mg/dl  Negative	50mg/dl  100mg/dl
GLUT	Negative	
CREA	100mg/dl  20mg/dl	0mg/dl  0mg/dl

## STEP 5

Read the results when all upper "Control" or "C" lines have appeared. Negative results can be read as soon as all "Test" or "T" lines are visible.



( Wait 5 minutes to determine a Positive Result )

### INTERPRETING RESULTS

NEGATIVE	NEGATIVE	POSITIVE	INVALID
A negative result is indicated by the presence of the lower "Test" or "T" Line of each designated drug.	The presence of even a very light "Test" or "T" Line indicates a negative result.	A positive result is indicated by the absence of the lower "Test" or "T" Line for a specific drug. Wait 5 minutes to determine a positive result.	An invalid result is indicated by the absence of the upper "Control" or "C" Line and the lower "Test" or "T" Line for a specific drug. If this occurs run a second test.